

# INFANT SCALE CALIBRATION PROCEDURE

\*\* You will need 50 lbs of CERTIFIED CALIBRATION WEIGHTS

1. Begin with the power off. While pressing and holding the KG/LB key press the **ON/ZERO** key to turn the scale on in "Function Mode". F 1 will appear on the display. Release the KG/LB key only after F 1 appears on the display.
2. Press the KG/lb key once. F 2 will show on the display.
3. Press the KG/LB key once to confirm Calibration Mode.
4. The display will read 9 or 16. If it reads 16 press the KG/LB key once. If the display reads 9 press the ON/ZERO key once and then press the KG/lb key.
5. The display should read **0 - -** then automatically advance to **2 - -**
6. Gently place a **50 lb** weight on the center of the scale platform.
7. After the weight has stabilized for 10 seconds press the **KG/LB** key. The display will show **F 1**. The calibration process is complete.
8. Remove the weight and press the **OFF** key to turn the power off.
9. Press the **ON/ZERO** key to turn on the power. Test the scale with a certified weight of **20 lbs**.

For technical support call **888-386-8225**

